





BRUKER MANUAL/USERS MANUAL NS 1500 COMFOR & STANDARD

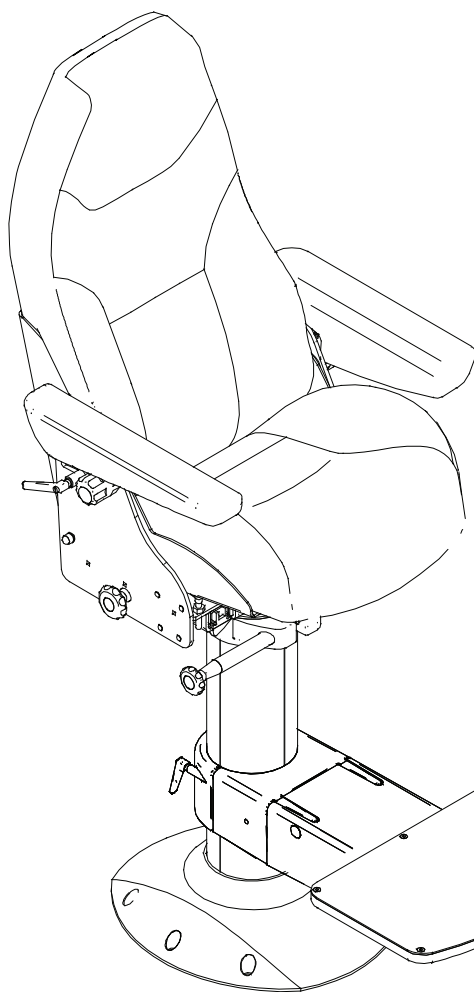

6400

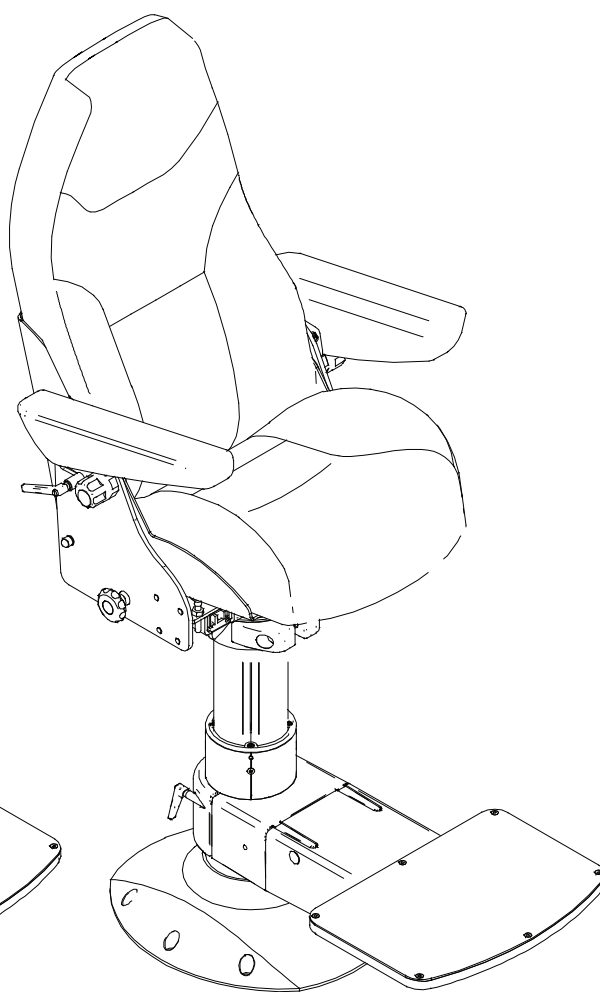

6500, 6510, 6520


FORHANDLER/DEALER.



NorSap as

Mjåvannsveien 35
4628 Kristiansand S
Norway

Tel +47 38 18 52 00
Fax +47 38 18 20 86
www.norsap.no

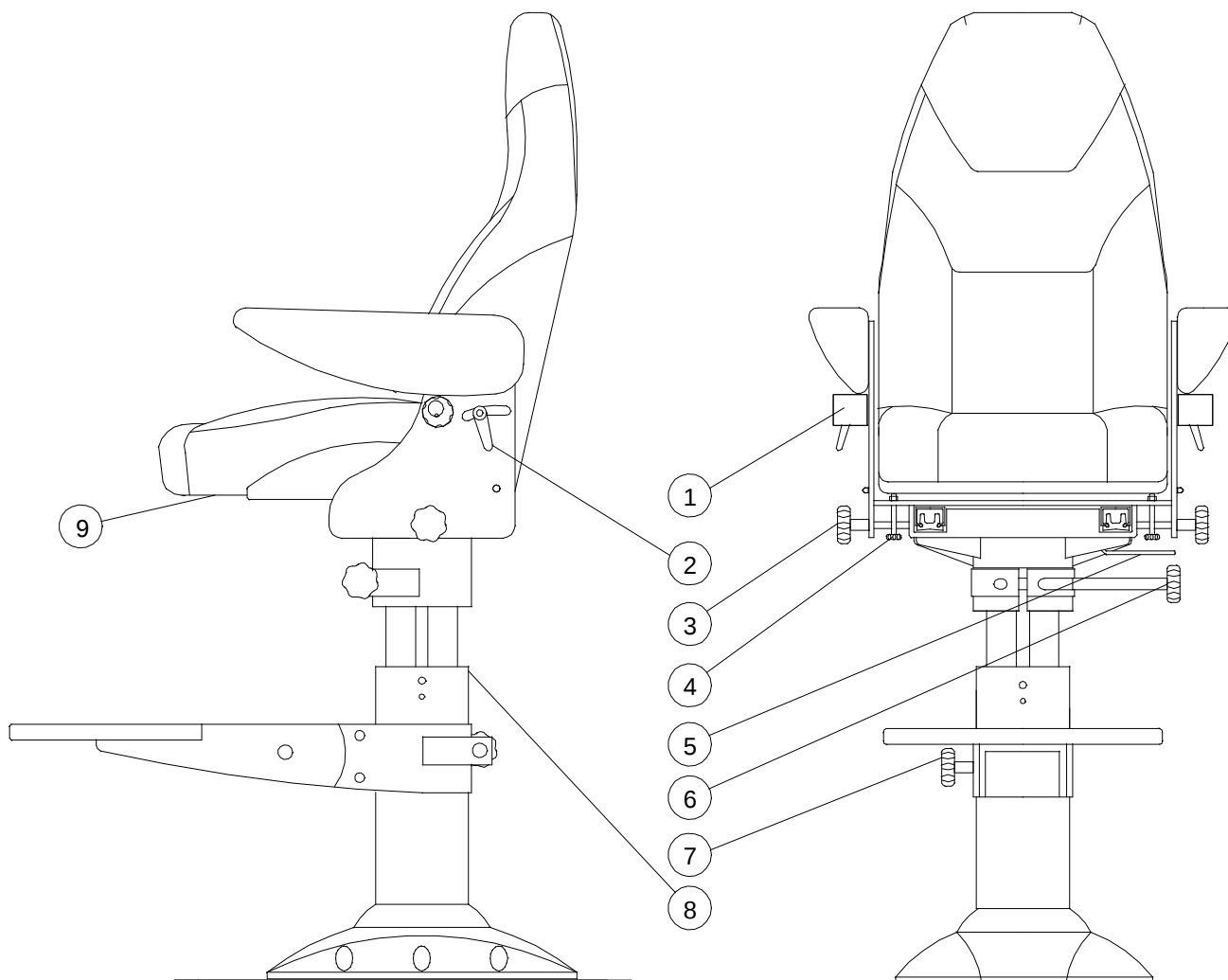

- ① JUSTERING AV EKSENTRISK ARMHVILER
- ② HENDEL FOR JUSTERING AV RYGGVINKEL
- ③ LÅSING AV TOPP GLIDESKINNE
- ④ REGULERING AV SETEVINKEL.
- ⑤ UTLØSERSPAK FOR HØYDEREGULERING (Inngår ikke i Standard modellen Art.nr.6400)
- ⑥ FASTLÅSING AV SVING
- ⑦ FASTLÅSING AV FOTHVILER
- ⑧ TELESKOPSØYLE (fast høyde på søyle til Standard modellen Art.nr.6400)
- ⑨ SETE - FREM OG TILBAKE SLEIDE

Solid underlag er nødvendig for sikker montering. Under lagring eller transport hender det at teleskopsøyle ⑧ kan sette seg i nedre stilling. Det kan med letthet trekkes opp ved å skru til skruen av sving ⑥, løft opp utløserhendelen ⑤ samtidig at hele stolen trekkes opp.

- ① ADJUSTABLE ECCENTRIC ARM REST
- ② ADJUSTABLE HANDLE FOR THE BACK REST ANGLE
- ③ LOCKABLE TOP GLIDE PLATE
- ④ REGULATING DEVICE FOR SEAT ANGLE
- ⑤ RELEASE HANDLE (not included on Standard model Art.No.6400)
- ⑥ LOCKABLE ARM FOR SWING
- ⑦ LOCKABLE ARM FOR FOOT REST
- ⑧ TELESCOPE COLUMN (Constant/fixed height on Standard model Art.No.6400)
- ⑨ SEAT - FORWARD LIFT AND SLIDE

A strong and level foundation is required to secure the installation of the base. It may occur that during storage or transport that the telescope column ⑧ fastens in the lower position. This can easily be adjusted by fastening the lockable arm for swing ⑥, lift up the release handle ⑤ at the same time pull up the chair.

VEKT/WEIGHT: 47KG

BRUKER MANUAL/USERS MANUAL NOR-SAP 1500



NorSap as

Mjåvannsveien 35
4628 Kristiansand S
Norway

Tel +47 38 18 52 00
Fax +47 38 18 20 86
www.norsap.no

Pleie og vedlikehold av lær

De fleste lærkvaliteter har fått en eller annen form for overflatebeskyttelse. Når så møblene blir tatt i bruk, tilfører man fett fra for eksempel hender og nakke. Dersom man ikke regelmessig rengjør og beskytter materialet på nytt, løser kroppsfettet langsomt opp all beskyttelse i overflaten. Uten slik beskyttelse trenger fett inn i læret og danner store, mørke fettflekker eller forårsaker avfarging. Rens med Soft Cleaner helst månedlig, bruk Leather Protection Cream 2-4 ganger pr. år. Vær nøye med regelmessig rengjøring og beskyttelse. Det er Du som bestemmer hvor lenge møbelet skal vare og hvor lenge den skal beholde sin eleganse.

- Beskytte læret mot direkte sollys over lengre tid.
- Varmekilde må ikke stå nærmere en 20-30 cm fra møbelet.
- Fjern støv og smuss fra læret regelmessig med Soft Cleaner en gang i måneden.
- Ta kontakt med oss eller nærmeste møbelhandler for mer rengjøringsmiddel eller rengjøringsdetaljer.

Vår leverandør på rensemiddel er:

Leather Master AS

6220 Straumgjerde
NORWAY
Tlf. +47 70 25 35 50
Fax. +47 70 25 35 58
Internet: leathermaster.no
E-mail: service@leathermaster.no

Care and maintenance of leather

Most leather qualities have originally received one or another type of surface protection treatment. When the furniture is taken in use, grease from hand and neck will be in contact with the leather on a regular basis. Without proper cleaning the leather will over a period of time leave grease spots or discoloration's. To avoid this clean with Soft Cleaner on a regular basis, we advise once a month. Use Leather Protection Cream or a leather cream with impregnation 2-4 times a year. This will protect against spots, increase wear resistance and preserve the leathers softness. By following these instructions you will ensure a long and lasting leather quality.

- Avoid direct sunlight over a longer period of time.
- Do not place a heat source within a distance of 20-30 cm.
- Remove dust or vacuum, use Soft Cleaner once a month.
- Contact us or an authorized leather producer or Furniture Dealer for more cleaning medium or for further cleaning details.

Our supplier of cleaning medium is:

Leather Master AS


6220 Straumgjerde
NORWAY
Tlf. +47 70 25 35 50
Fax. +47 70 25 35 58
Internet: leathermaster.no
E-mail: service@leathermaster.no



NorSap as

Mjåvannsveien 35
4628 Kristiansand S
Norway

Tel +47 38 18 52 00
Fax +47 38 18 20 86
www.norsap.no

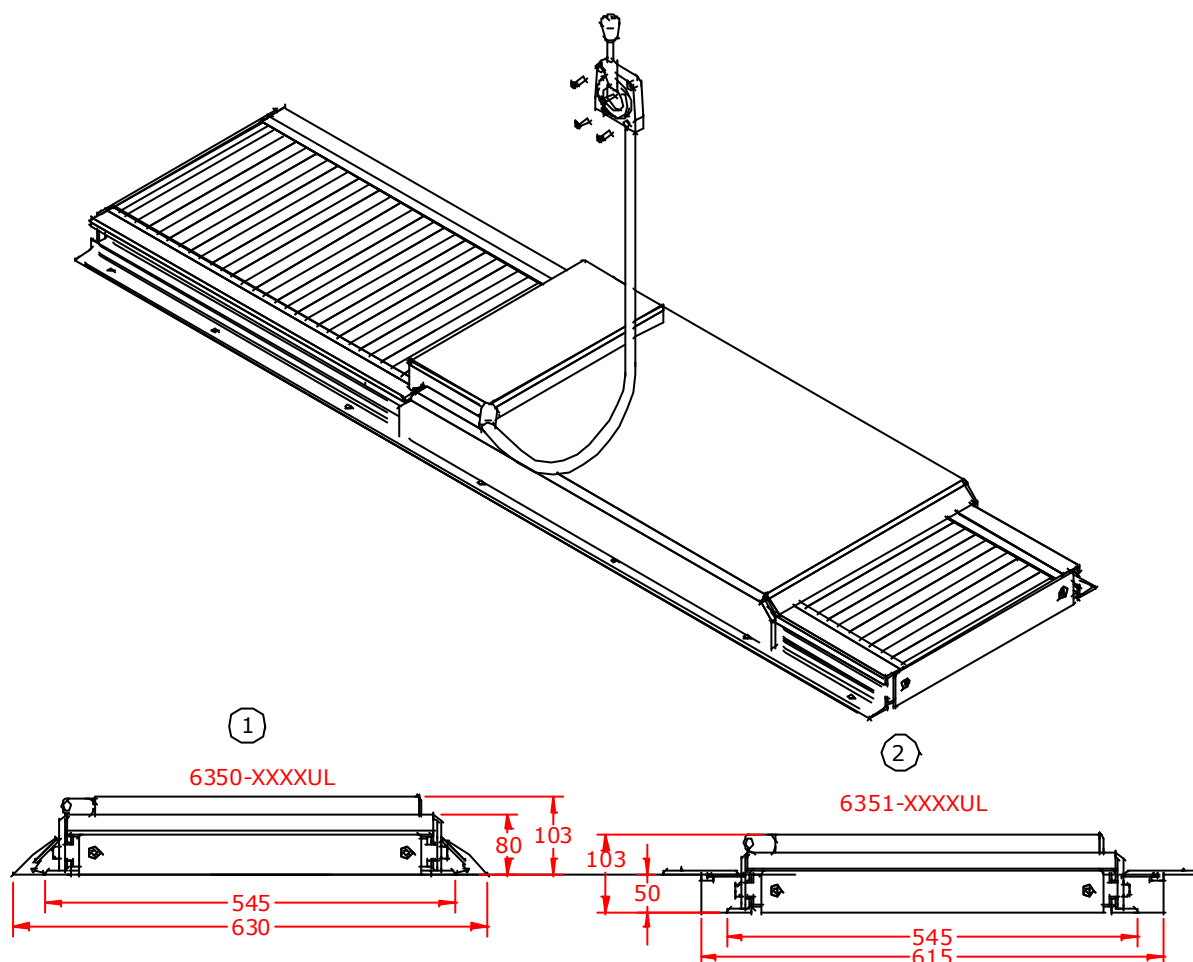

① DØRKSINNE PÅMONTERT
ANTI-SNUBLE LIST MED BØRSTE.

② DØRKSINNE NEDFELT OG PÅMONTERT
KANTPROFIL MED BØRSTE.

① ANTI-TRIP SKIRTING BOARD
DECK RAIL WITH BRUSHES

② FLUSH MOUNTED DECK RAIL
WITH BRUSHES.

MONTERING:

NB! VIKTIG

Underlaget må være plant og ha godt feste for skruer. Glideskinne/hjulspor må være fri for metallrester eller andre små deler.

**IKKE BRUK SMØREFETT I HJUL
SPORENE. DETTE VIL TILTREKKE
UØNSKET PARTIKLER OG HINDRE
HJULENE.**

MOUNTING

NB! IMPORTANT

A strong and level foundation is required to secure the installation of the deck rail. Gliding/wheel tracks must be free from scraps of metal or other items.

**DO NOT USE GREASE INSIDE THE WHEEL
TRACKS. THIS WILL ATTRACT UNDESIRABLE
PARTICLES AND OBSTRUCT THE WHEELS.**


BRUKER MANUAL / USERS MANUAL
GLIDESKINNE / DECKRAIL
Art. No. 6350-XXXXUL
Art. No. 6351-XXXXUL



NorSap as

Mjåvannsveien 35
4628 Kristiansand S
Norway

Tel +47 38 18 52 00
Fax +47 38 18 20 86
www.norsap.no

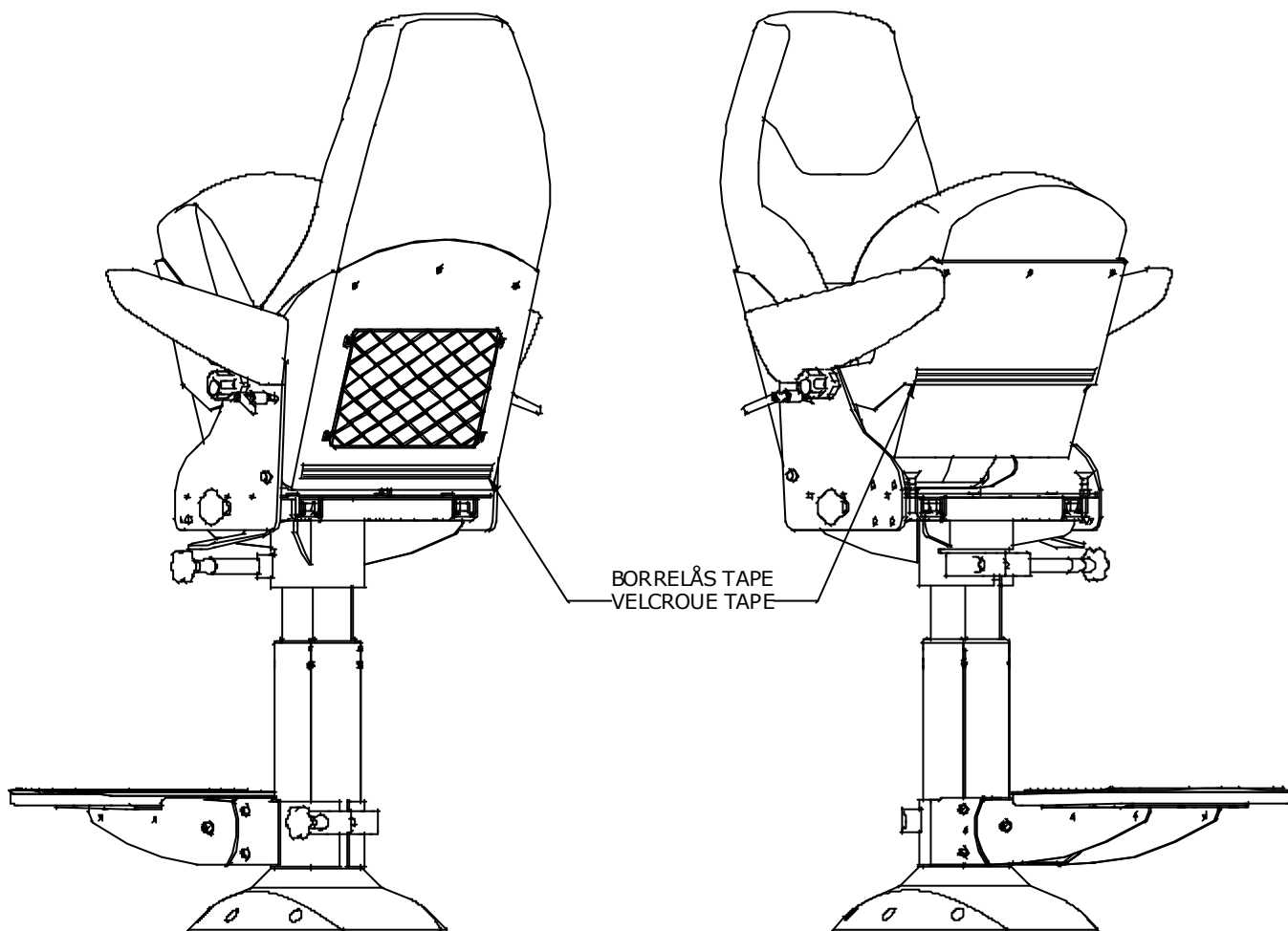

FESTE FOR SETETREKK:

For å sikre setetrekket, settes borrelås nederst på ryggen og under setet som illustrert ovenfor.

Art.nr.6470 Setetrekk m/amlen i stoff.
Art.nr.6451 Setetrekk m/amlen i vannavstøtende.

SECURING SEAT COVER:

To secure seat cover, place velcroue tape on the lower back rest and underneath the seat as illustrated above.

Art.no.6470 Seat cover w/armrest in fabric.
Art.no.6451 Seat cover w/armrest in water repellent.

SECURING SEAT COVER
NS1500 PILOT CHAIR

FESTE FOR SETETREKK TIL
NS1500 STOL



NorSap as

Mjåvannsveien 35
4628 Kristiansand S
Norway




Tel +47 38 18 52 00
Fax +47 38 18 20 86
www.norsap.no

Filename:


S:\Tekinfo\NS1500 BM\6500-06.dft

Date

25.02.00.

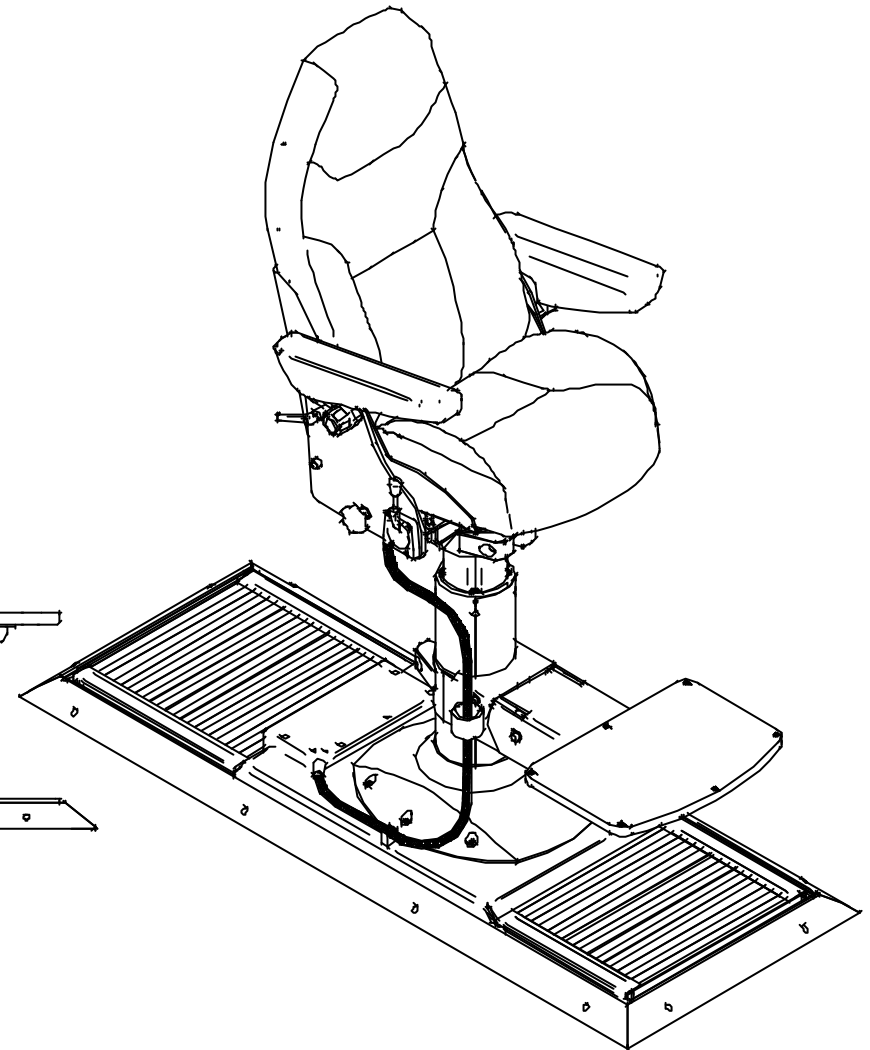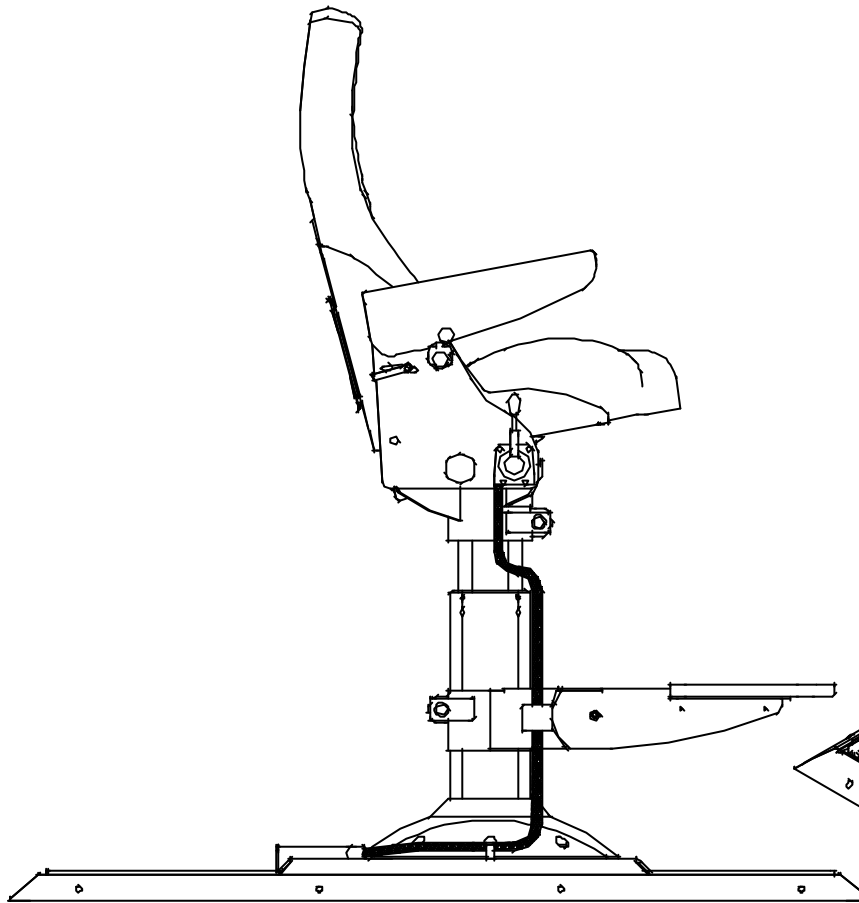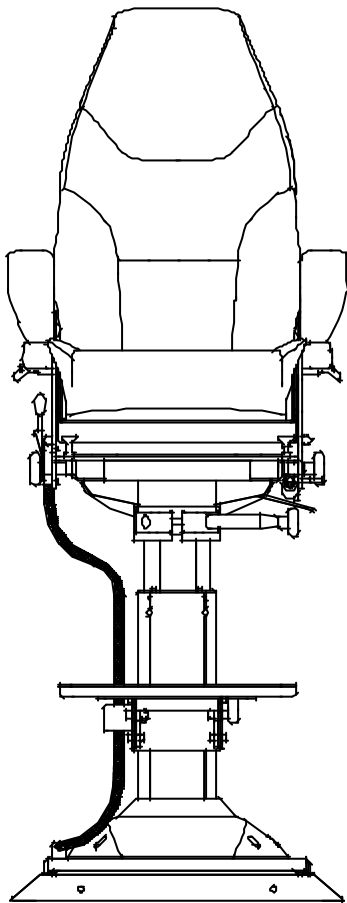

REV:	REVISION ISSUED FOR:	Date	DRN. Sign.	Date Chk.	Date Approved	Nor SapTM NorSap as Mjåvannsvæien 35 Tel +47 38 18 52 00 4628 Kristiansand S Fax +47 38 18 20 86 Norway www.norsap.no	
A-2	FOR INFORMATION	08.01.02	Grethe			Title: NOR SAP 1500 MOUNTING OF LOCK FROM SITTING POSITION	
A-1	FOR INFORMATION	06.06.01	Grethe				
Filename:			Scale:	Mod./Vers.	Drp.no.	Rev.	
S:\Tekinfo\Ns1 500 BM\6500+6350-1500UL.dft			N\A	V02	6500+6350-1500UL	A-2	

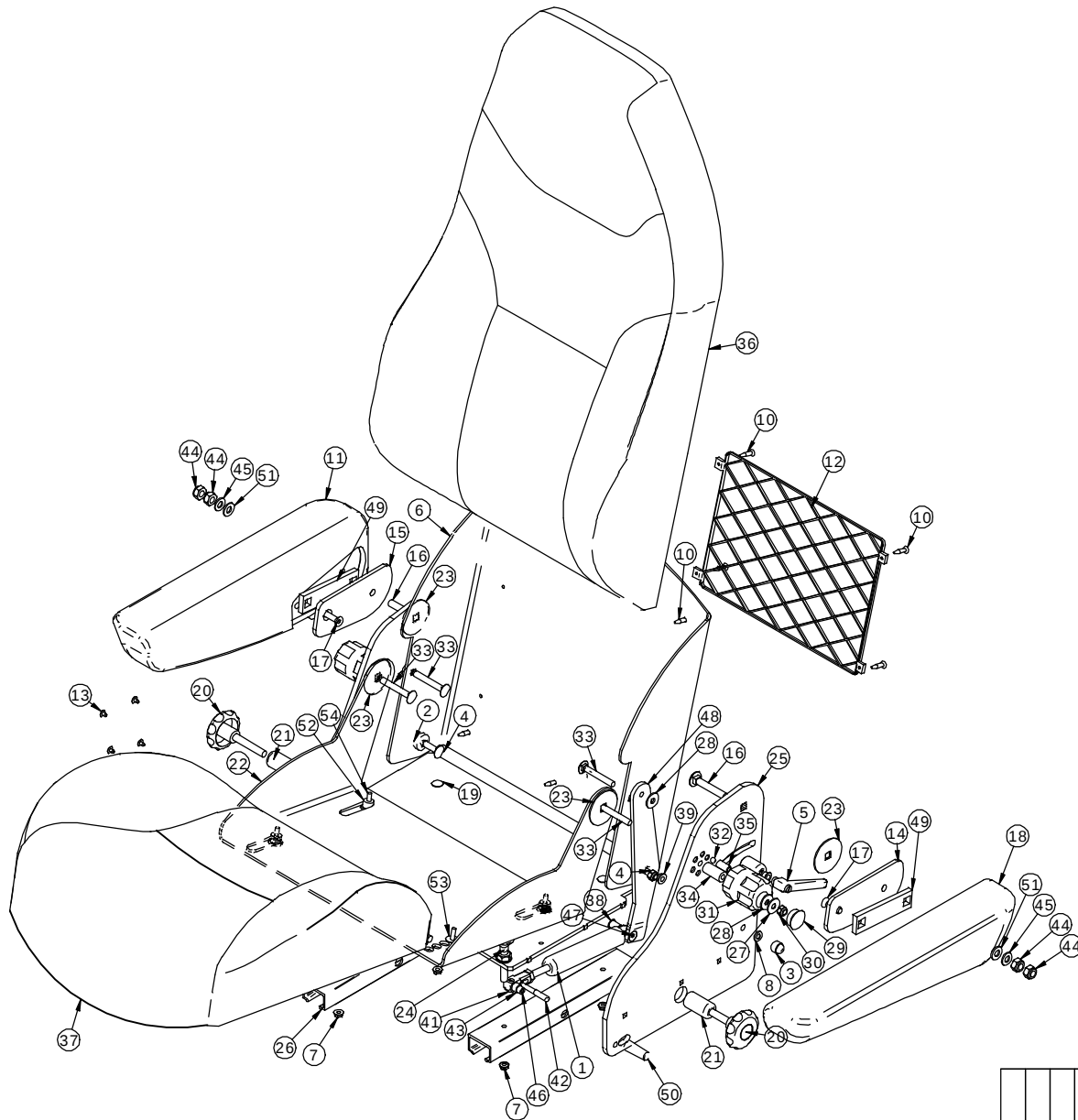

R-0010 SEAT FRAME NS1500 B1,B2,B3 98A PILOT CHAIR


Spare part nr.	Item name	Qty.	Pos.Nr.
GAS-021	GAS SUSPENSION KOB1K-3-055-215/600N	1	1
PLA-0501	DISTANCE PIECE ø25x8,2x5	2	2
B757-M8S	HEX NUT PROTECTION CAP M8 BLACK DIN 1587	2	3
B413-M8X30	ROUND HEAD SQ.NECK CARR.BOLT M8x30	2	4
RATT-004	HAND REGULATOR M8	2	5
SAP-0087	BACK REST PLATE	1	6
B463-M6	HEX.NUT M6 SELFLOCK DIN6923	4	7
B481-M8	ROUND WASHER FLAT M8 DIN125A	5	8
MP145	EXTENSION PIECE ø20x8,5x30 BLACK	2	9
B434-4,2x25	COUNTERS.SELF.TAPP.SCR. 4,2x25 A4	10	10
DEL-128H	ARMREST NS1500 RIGHT SIDE	1	11
BES04M10X52	SAFEKEEPING NET BLACK	1	12
PLA-11D4	COVER PLUG ø4 BLACK	4	13
SAP-0089V	HINGE FOR ARMREST LEFT SIDE 6mm	1	14
SAP-0089H	HINGE FOR ARMREST RIGHT SIDE 6mm	1	15
B413-M10X60	ROUND HEAD SQ.NECK CARR.BOLT M10X60	2	16
B420-M8X30	HEX.COUNTERS. HEAD SCR. M8x30 DIN7991	2	17
DEL-128V	ARMREST NS1500 LEFT SIDE	1	18
B413-M6X20	ROUND HEAD SQ.NECK CARR.BOLT M6X20	4	19
RATT-029	GRIP HANDLE ø60 M10x60	2	20
SAP-0051	PRESSURE CASING	2	21
SAP-0090	SEAT PLATE	1	22
SAP-0009	ROUND WASHER FLAT ø55x2	4	23
STL-001	HEIGHT ADJ.SCREW Ø30 M10x52	2	24
DEL-098	SEAT FRAME NS1500	1	25
DEL-125	GLIDE RAIL HOLDER	2	26
B480-M8	ROUND WASHER FLAT M8 (8,4x25x2)	2	27
B780-M8	ROUND WASHER FLAT NYLON M8 DIN 9021	3	28
PLA-0701	COVER PLUG INSERT ø25	2	29
B462-M8	HEX. NUT NYLOC M8 DIN 985	2	30
SAP-0084	ECCENTRIC REGULATOR	2	31
PLA-0901	NYLON BALL ø8	2	32
B413-M8X60	ROUND HEAD SQ.NECK CARR.BOLT M8X60	4	33
SAP-0085	SPACER ROUND ø16	2	34
FJR-001	SPRING ø10x22x1,5	2	35
PUTE-001	BACK REST NS1500	1	36
PUTE-002	SEAT REST NS1500	1	37
B472-M8K	POPNUT M8 8X16	1	38
PLA-028X12X16	COVER PLUG WITH COLLAR 8X12X16X2 PA66	1	39
B463-M8	NUT SELFLOCK M8	2	40
GASDEL-002	RELEASE FITTING FOR GAS SUSPENSION	1	41
GASDEL-003	RELEASE STICK FOR REGULATING BACK REST	1	42
SAP-1055	RANGE CASING FOR GAS SUSPENSION	1	43
B462-M10	HEX NUT M10 NYLOC	4	44
B481-M10	ROUND WASHER FLAT M10	2	45
B424-M10X35	HEX.SOCKET SET SCR.W/FLAT POINT M10X35	1	46
B420-M8X20	HEX.COUNTER.HEAD SCR.M8X20 DIN7991	1	47
SAP-0781	REG.PLATE FOR BACKREST	1	48
SAP-1292	BUSHING PLATE 8mm	2	49
PLAST-040	PEG COVER 8x40	1	50
B781-M10	ROUND WASHER FLAT M10 NYLON DIN125	2	51
SAP-1759	SLIDER KNOB	3	52
SAP-1758	LOCKING KNOB	1	53
B415-M6X20	HEX SOCKET CAP SCREW M6X20	4	54

Dato / Date	Tegn. / Drw.		Kontr. / Check	Målestokk / Scale 1:10	 NorSap as Mjåvannsveien 35 4628 Kristiansand S Norway Tel +47 38 18 52 00 Fax +47 38 18 20 86 www.norsap.no
	Sign. / Sign.				
	Filnavn / Filename: S:\Tekinfo\Ns1500 BM\R-0010.dft		Tegnrnr. / Drgnr. R-0010		




NorSap as
Mjåvannsveien 35
4628 Kristiansand S
Norway
Tel +47 38 18 52 00
Fax +47 38 18 20 86
www.norsap.no

Tittel / Title:
**SEAT FRAME NS1500 B1,B2,B3
AND STANDARD MOD.2000A
ASSEMBLY**

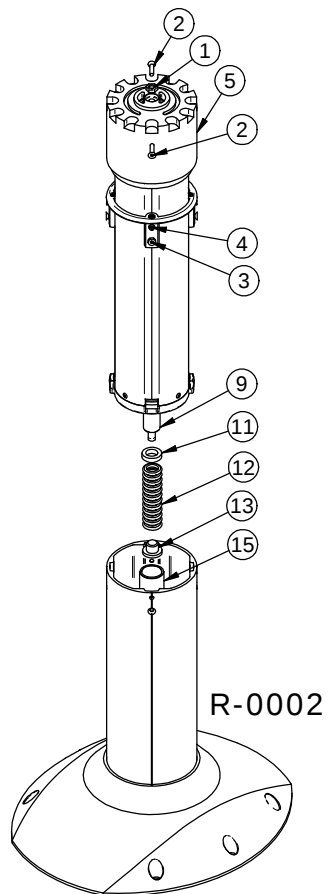


R-0002

(R-0002expBM.cfg)

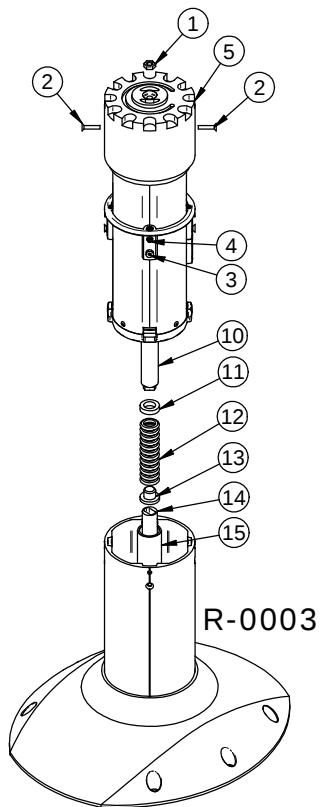

R-0003

(R-0003expBM.cfg)

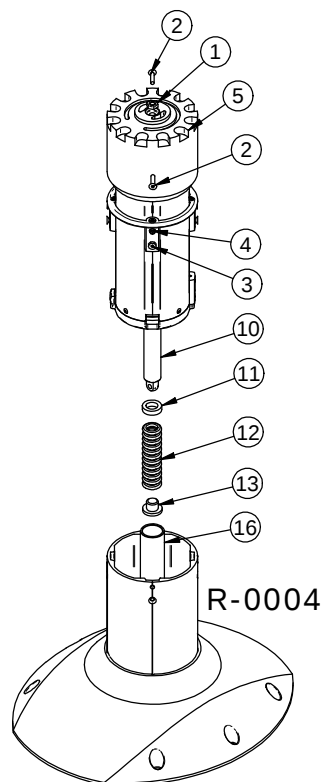

R-0004


(R-0002expBM.cfg)

R-0002, R-0003, R-0004 TELESCOPIC COLUMN NS1500 B1,B2,B3 V-02 PILOT CHAIR			
Spare part nr.	Item name	Qty.	Pos.Nr.
B361-M10	HEX.NUT M10 FOR GAS SUSPENSION	1	1
B420-M6X30	HEX.COUNTERSUNK HEAD SCREW M6x30	2	2
B420-M6X20	HEX.COUNTERSUNK HEAD SCREW M6x20	4	3
B424-M8X8	HEX.SOCK.SET SCR.w\FLST POINT M8x8 DIN913	4	4
MP151	CAST BUSHING NS1500 INNER TUBE	1	5
R-0002	SPARE COLUMN FOR 6500 (B1)	1	6
R-0003	SPARE COLUMN FOR 6510 (B2)	1	7
R-0004	SPARE COLUMN FOR 6520 (B3)	1	8
GAS-001	GAS SUSPENSION FOR 6500 (B1)	1	9
GAS-002	GAS SUSPENSION FOR 6520 (B3) AND 6510 (B2)	1	10
SAP-1559	BUSHING	1	11
FJR-009	SPIRAL SPRING L=120mm	1	12
SAP-1558	PLUG	1	13
SAP-0796-090	EXTENSION PIECE L=90mm	1	14
SAP-1157	EXTENSION TUBE FOR GAS SPRING L=400mm	1	15
SAP-0149	EXTENSION TUBE FOR GAS SPRING L=300mm	1	16



Dato				Målestokk / Scale
				1:10
Sign.	Tegn. / Drw.	Kontr. / Check	Godkjent for produksjon / Approved for production	
	Dato / Date	22.04.2004		
Filnavn / Filename			Tittel / Title	
S:\Tekinfo\Ns1500 BM\R-0002.dft			TELESCOPIC COLUMN NS1500 B1,B2,B3 V-02 ASSEMBLY	
			Tegn.nr.	Verj / Rev
			R-0002	



NorSap as
Mjåvannsvelen 35 | Tel +47 38 18 52 00
4628 Kristiansand S | Fax +47 38 18 20 86
Norway | www.norsap.no

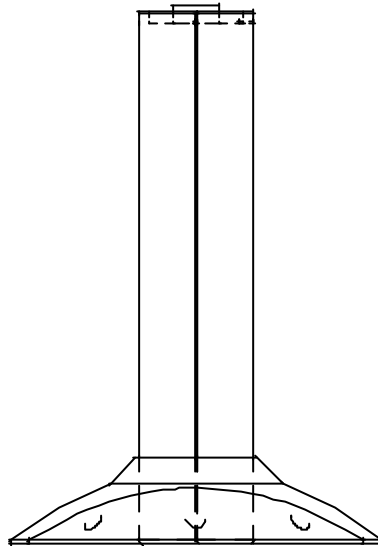


R-0007

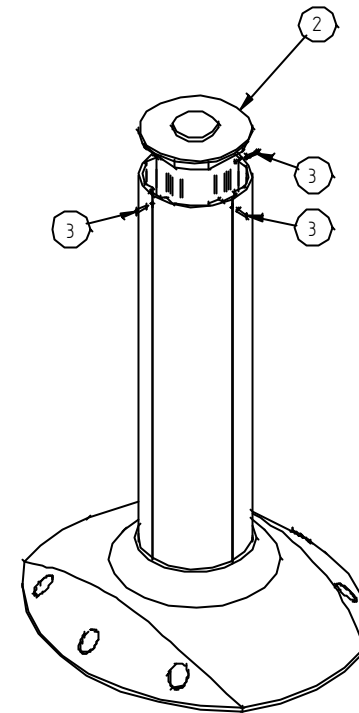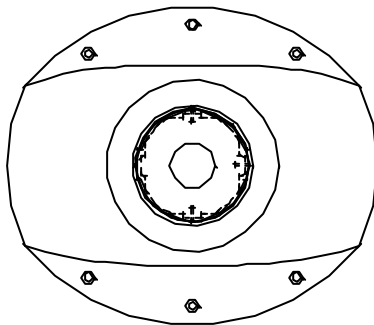

Item nr.	Item name	Qty.	Pos.Nr.
R-0007	SPARE COLUMN FOR NS1500 STANDARD Art.No.6400	1	1
SAP-0101	THRUST PLATE	1	2
B290-3X18	PIN SPRING Ø3x18	3	3




		Dato				Målestokk / Scale	 NorSap as Mjåvannsvæien 35 4628 Kristiansand S Norway www.norsap.no	Tel +47 38 18 52 00 Fax +47 38 18 20 86
						1:10		
				Tegn. / Drw.	Kontr. / Check	Godkjent for produksjon / Approved for production		
		Sign.	Dato / Date	00.01.27			Tittel / Title	
			Sign. / Sign.	Grethe			COLUMN FOR NS1500 STANDARD Art.No.6400 ASSEMBLY	
			Filnavn / Filename: S:\Tekinfo\Ns1500_bmlR-0007.dft					Tegn.nr. / Dwg.nr. R-0007

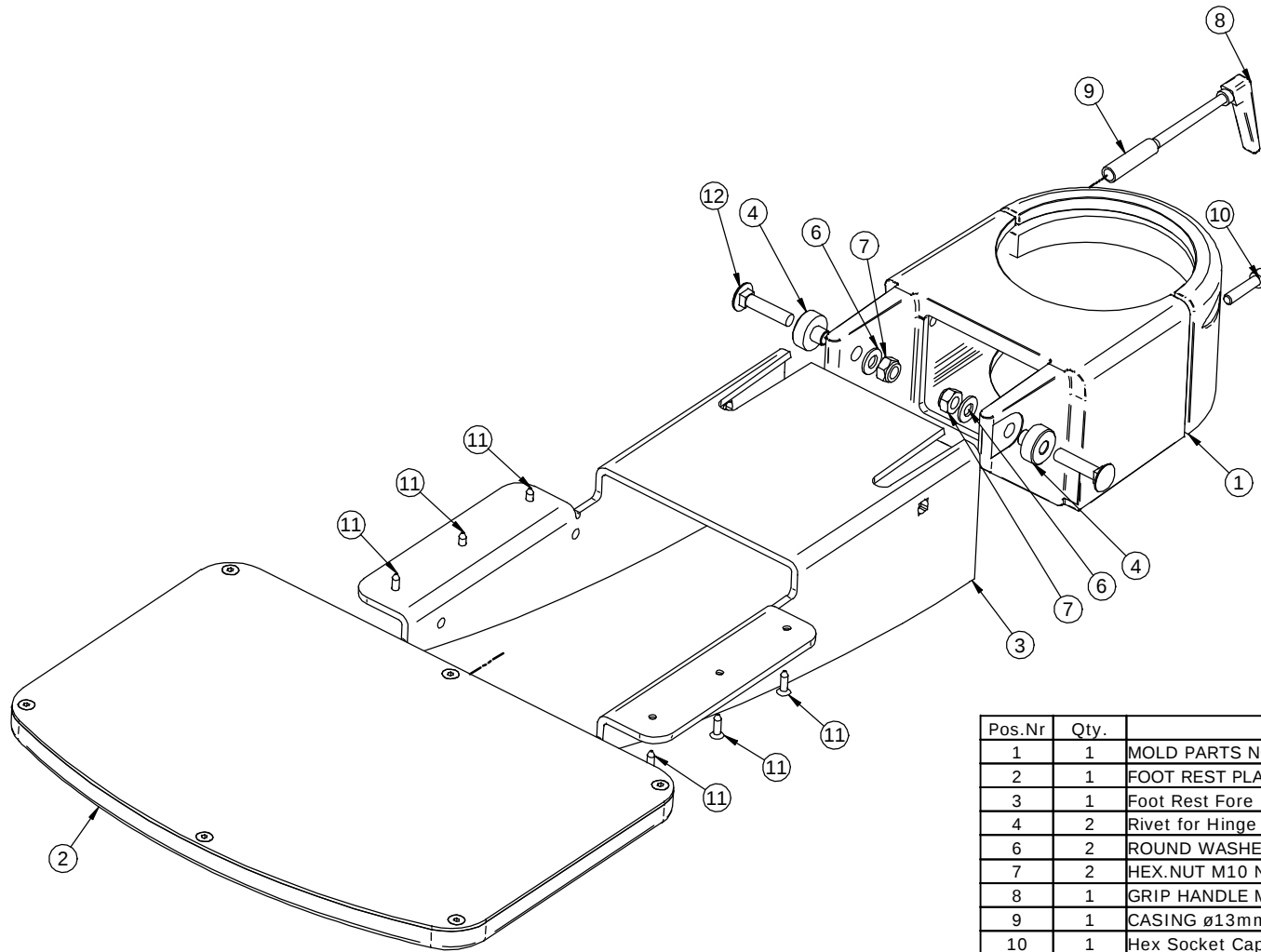


Pos.Nr	Qty.	Item Description	Item Nr.	Rev.
1	1	MOLD PARTS NORSAP 1500 FOTREST	DEL.687	A-1
2	1	FOOT REST PLATE	DEL-136	A-2
3	1	Foot Rest Fore	SAP-1566	A-1
4	2	Rivet for Hinge	SAP-1822	A-1
6	2	ROUND WASHER FLAT M10 NYLON DIN125	B781-M10	
7	2	HEX.NUT M10 NYLOC	B462-M10	
8	1	GRIP HANDLE M8x80	RATT-018-80	A-1
9	1	CASING Ø13mm L=51mm	SAP-0576_51	A-1
10	1	Hex Socket Cap Screw M8x35	B415-M8X35	
11	6	RAISED COUNTERS.SELF TAPP.SCR.5,5x19	B432-5.5X19	
12	2	ROUND HEAD SQ.NECK CARR.BOLT M10X45	B413-M10X45	

Del-085exp1.cfg

Dato				Målestokk / Scale	n/a
		Tegn. / Drw.	Kontr. / Check	Godkjent for produksjon / Approved for production	
Sign.	Dato / Date	28.04.04.		Nor Sap ™ NorSap as Mjåvannsvæien 35 Tel +47 38 18 52 00 4628 Kristiansand S Fax +47 38 18 20 86 Norway www.norsap.no	
	Sign. / Sign.	Harald			
Filnavn / Filename: S:\Tekinfo\NS1500 BM\DEL-085.dft					Tittel / Title: FOOT REST CASTED for NS1500 B1,B2,B3 AND STANDARD ASSEMBLY Tegn.nr. / Drgnr.: DEL-085 Versj / Rev.: A-6

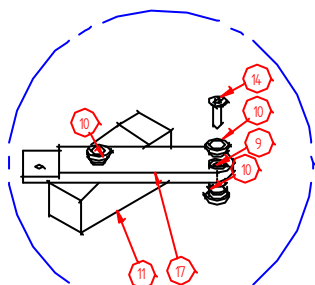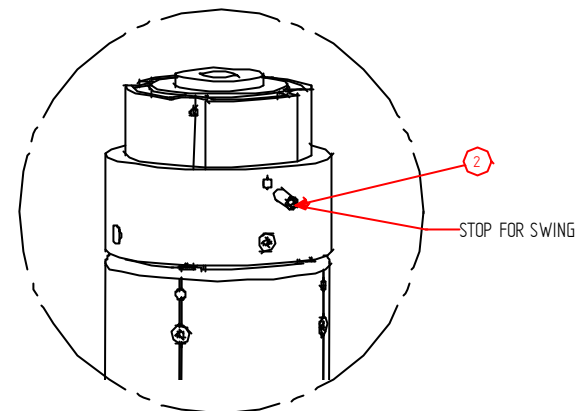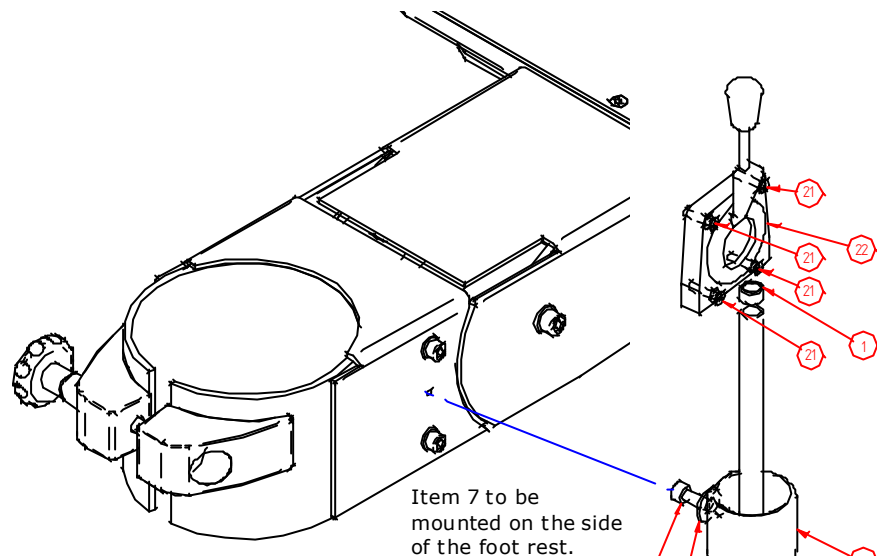


DETAIL A-A

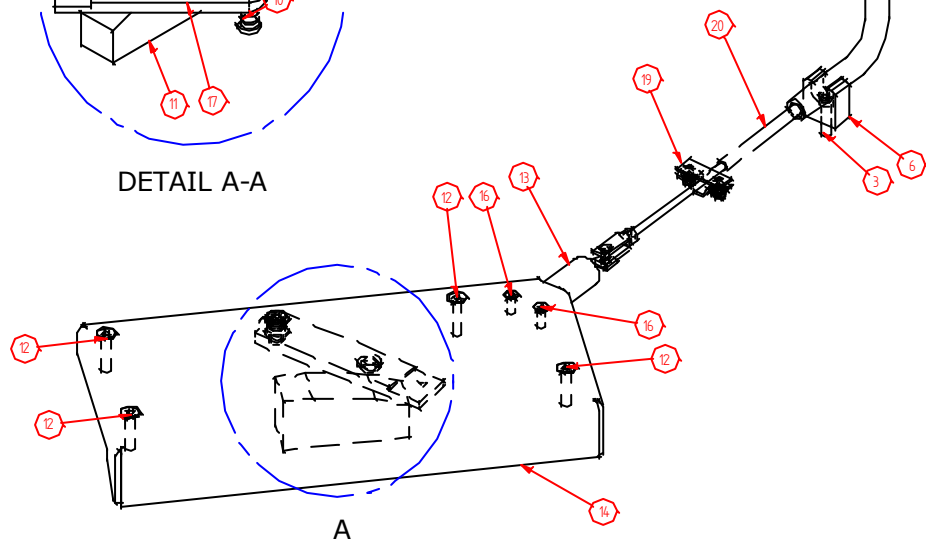

6375

6375 RELEASE FROM SITTING POSITION NS1500			
Spare part nr.	Item name	Qty.	Pos.Nr.
SAP-0322	PROTECTION CAP	1	1
B424-M8X25	HEX.SOCKET HEAD SCREW M8X25 DIN916	1	2
B415-M6X20	HEX.SOCKET CAP SCREW M6X20	1	3
B415-M8X30	HEX.SOCKET CAP SCREW M8X30	1	4
B480-M8	WASHER ROUND FLAT M8	1	5
PLA-12D16	FLEXIBLE TUBE HOLDER	1	6
SAP-0545	FLEXIBLE TUBE GUIDER	1	7
SLA-002	FLEXIBLE TUBE	1	8
DEL-042 CASING FOR BRAKE SYSTEM			
SAP-0559	HINGE BOLT	1	9
PLA-0301	TUBE SLEEVE WITH COLLAR ø10x12x16	3	10
SAP-0546	BRAKE BLOCK	1	11
B420-M6X30	SOCKET COUNTERS.HEAD SCR.M6x30	4	12
SAP-0294	CASING FOR FLEXIBLE TUBE	1	13
B420-M6X20	HEX.COUNTERSUNK HEAD SCREW M6x20	1	14
SAP-0353H	CASING FOR BRAKE	1	15
B420-M5X16	HEX COUNTERSUNK HEAD SCREW M5X16	2	16
SAP-0287H	BRAKE ARM PLATE	1	17
DEL-048 FORK ,WIRE AND SADDLE			
WIRE003	FORK FOR WIRE	1	18
WIRE002	WIRE SADDLE	1	19
WIRE004	CONTROL CABLE WITH NUT	1	20
DEL-047 JOYSTICK FOR BRAKE			
B420-M5X25	HEX COUNTERSUNK HEAD SCREW M5X25	4	21
SPAK-001	HANDLE FOR BRAKE	1	22

		Målestokk / Scale:		1:5		 NorSap as Mjåvannsveien 35 Tel +47 38 18 52 00 4628 Kristiansand S Fax +47 38 18 20 86 Norway www.norsap.no	
		Tegn. / Drw.		Godkjent for produksjon /			
		Dato / Date		22.11.00.		Tittel / Title: RELEASE FROM SITTING POSITION FOR NS 1500 ASSEMBLY	
		Sign. / Sign.		GRE			
		Filnavn / Filename: S:\Tekinfo\NS1500 BM\6375.dft				Tegn. / Drw.: 6375	Versj. / Rev.: A - 1

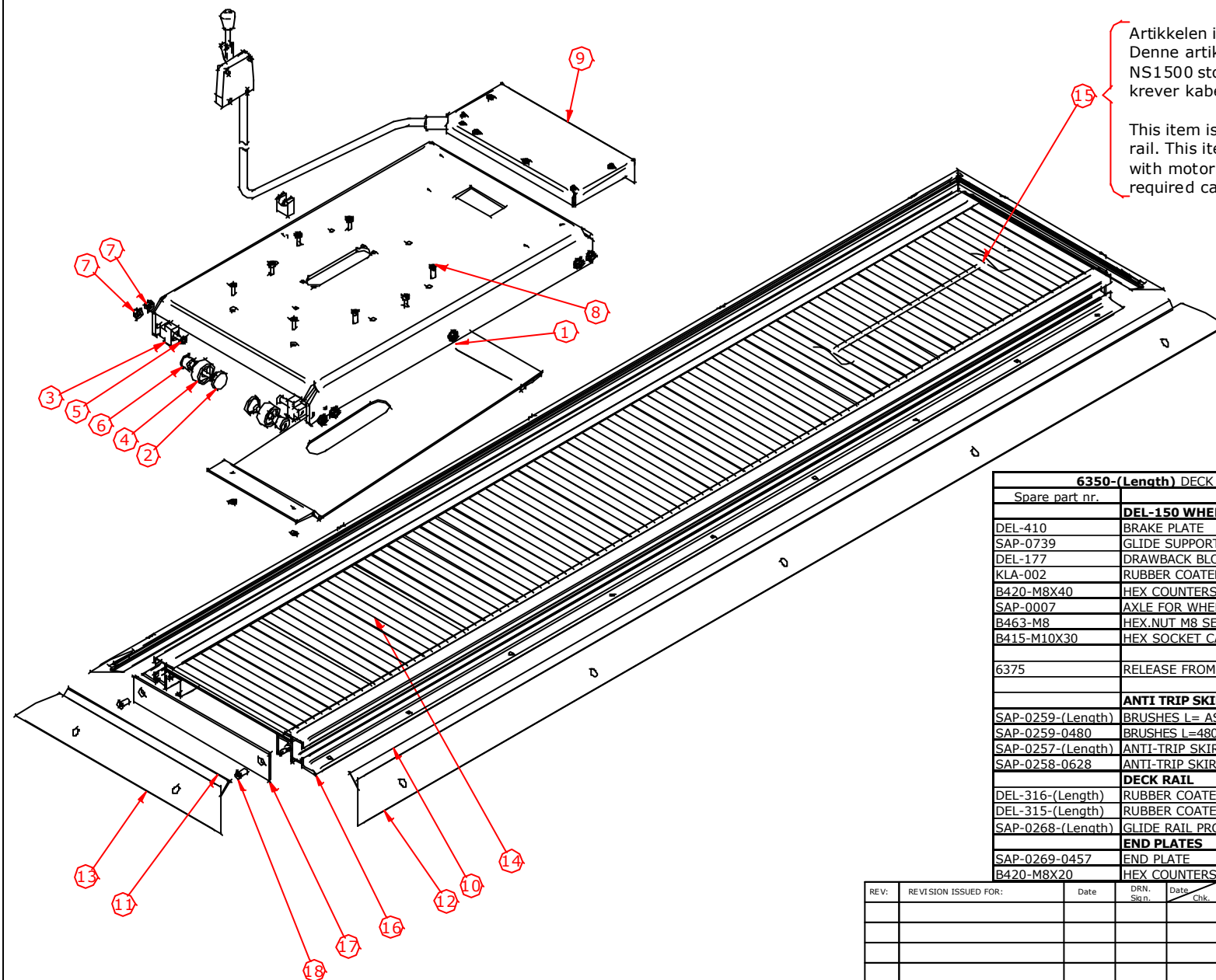


Artikkelen inngår ikke i standard dørskinne.
Denne artikkelen brukes sammen med
NS1500 stol med motor og for stoler som
krever kabel gjennomgang i dørk.

This item is not included on the standard deck
rail. This item is used by NS1500 pilot chair
with motor driven carriage and for chairs with
required cable passage through deck rail.

6350-(Length) DECK RAIL WITH ANTI TRIP SKIRTING BOARD NS1500			
Spare part nr.	Item name	Qty.	Pos.Nr
DEL-150 WHEEL CARRIAGE COMPLETE			
DEL-410	BRAKE PLATE	1	1
SAP-0739	GLIDE SUPPORT CASTORS	4	2
DEL-177	DRAWBACK BLOCK WITH FILT	4	3
KLA-002	RUBBER COATED WHEELS WITH BALL BEARINGS	6	4
B420-M8X40	HEX COUNTERSUNK HEAD SCREW M8X40	10	5
SAP-0007	AXLE FOR WHEEL	6	6
B463-M8	HEX.NUT M8 SELFLOCK DIN6323	10	7
B415-M10X30	HEX SOCKET CAP SCREW M10X30	6	8
6375	RELEASE FROM SITTING POSITION	1	9
ANTI TRIP SKIRTING BOARDS			
SAP-0259-(Length)	BRUSHES L= AS REQUIRED BY CUSTOMER	2	10
SAP-0259-0480	BRUSHES L=480mm	2	11
SAP-0257-(Length)	ANTI-TRIP SKIRT.BOARD L=AS REQ. BY CUSTOMER	2	12
SAP-0258-0628	ANTI-TRIP SKIRTING BOARD L=628mm	2	13
DECK RAIL			
DEL-316-(Length)	RUBBER COATED MIDDLE PLATE L=AS REQ.BY CUSTOMER	1	14
DEL-315-(Length)	RUBBER COATED MIDDLE PLATE WITH SLOT L=AS REQ.		15
SAP-0268-(Length)	GLIDE RAIL PROFILE L=AS REQ. BY CUSTOMER	2	16
END PLATES			
SAP-0269-0457	END PLATE	2	17
B420-M8X20	HEX COUNTERSUNK HEAD SCR.M8X20 DIN7991	4	18

REV:	REVISION ISSUED FOR:	Date	DRN. Sgn.	Date Chk.	Date Approved	 NorSap as Mjåvannveien 35 Tel +47 38 18 52 00 4628 Kristiansand S Fax +47 38 18 20 86 Norway www.norsap.no	
						Title:	
						DECK RAIL FOR NS1500 WITH ANTI-TRIP SKIRTING BOARD AND RELEASE FROM SITTING POSITION ASSEMBLY	
A-1	Issued for information	15.11.00	Grethe				
Filename:			Scale:		Mod./Vers.	Drq.no.	Rev.
S:\Tekinfo\NS1500 BM\6350-XXXXUL.dft			none		V-01	6350-(Length)UL	A-2

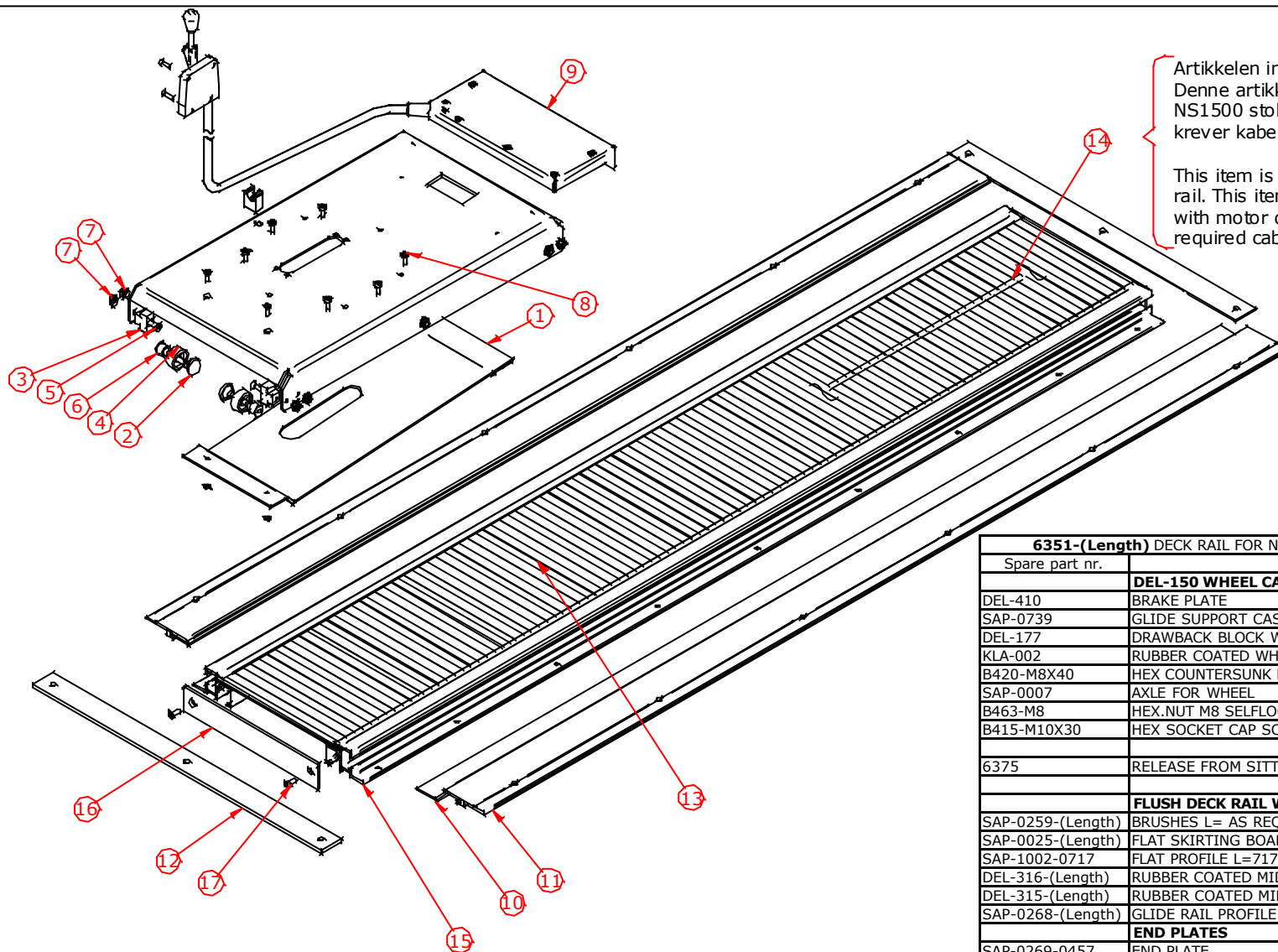

Artikkelen inngår ikke i standard dørskinne.
Denne artikkelen brukes sammen med
NS1500 stol med motor og for stoler som
krever kabel gjennomgang i dørk.

This item is not included on the standard deck
rail. This item is used by NS1500 pilot chair
with motor driven carriage and for chairs with
required cable passage through deck rail.

6351-(Length) DECK RAIL FOR NS1500 WITH FLUSH MOUNTED SKIRTING BOARDS			
Spare part nr.	Item name	Qty.	Pos.Nr.
DEL-150 WHEEL CARRIAGE COMPLETE			
DEL-410	BRAKE PLATE	1	1
SAP-0739	GLIDE SUPPORT CASTOR	4	2
DEL-177	DRAWBACK BLOCK WITH FILT	4	3
KLA-002	RUBBER COATED WHEEL WITH BALL BEARING	6	4
B420-M8X40	HEX COUNTERSUNK HEAD SCREW M8X40	10	5
SAP-0007	AXLE FOR WHEEL	6	6
B463-M8	HEX.NUT M8 SELFLOCK DIN6323	10	7
B415-M10X30	HEX SOCKET CAP SCREW M10X30	6	8
6375	RELEASE FROM SITTING POSITION	1	9
FLUSH DECK RAIL WITH SKIRTING BOARDS			
SAP-0259-(Length)	BRUSHES L= AS REQUIRED BY CUSTOMER	2	10
SAP-0025-(Length)	FLAT SKIRTING BOARD PROFILE L=AS REQ.BY CUST.	2	11
SAP-1002-0717	FLAT PROFILE L=717mm	2	12
DEL-316-(Length)	RUBBER COATED MIDDLE PLATE L=AS REQ.BY CUSTOMER	1	13
DEL-315-(Length)	RUBBER COATED MIDDLE PLATE WITH SLOT L=AS REQ.BY CUST.	1	14
SAP-0268-(Length)	GLIDE RAIL PROFILE L=AS REQ. BY CUSTOMER	2	15
END PLATES			
SAP-0269-0457	END PLATE	2	16
B420-M8X20	HEX COUNTERSUNK HEAD SCR.M8X20 DIN7991	4	17

REV:	REVISION ISSUED FOR:	Date	DRN. Sign.	Date	Chk.	Date	Approved	 NorSap as Mjåvannsvæien 35 Tel +47 38 18 52 00 4628 Kristiansand S Fax +47 38 18 20 86 Norway www.norsap.no	
								Title : DECK RAIL FOR NS1500 WITH FLUSH SKIRTING BOARDS AND RELEASE FROM SITTING POSITION ASSEMBLY	
A-1	Issued for information	14.11.00	Grethe					File name: S:\Tekinfo\NS 1500 BM\6351-XXXXUL.dft	
								Scale:	
								Mod./ Vers.	
								Drq.no.	
									Rev.
								6351-(Length)UL	A-2